LIFEWAVE®

# icewave onde glacée

## **Instructions**

Mode d'emploi

## INSTRUCTIONS FOR USE


### BEAD INSTRUCTIONS

Place one white and one tan patch on the body, using one of the locations shown. Apply the patches to clean, dry, undamaged skin. Patches may be worn for up to 12 hours before discarding. Remove the liner from the adhesive backing using the adhesive side of the patch to secure patch placement. Only remove half of the patch backing until an effective placement site has been located.

#### STEP 1

First, locate the acupressure point you wish to stimulate.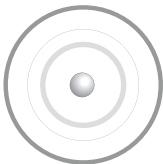

#### STEP 2

Next, remove a plastic bead from the bag. Remove the backing from the adhesive plastic patch included. Place the plastic bead in the center of the patch on the adhesive side of the patch.

#### STEP 3

Lastly, apply the bead and patch to the acupressure point, smoothing the patch on so that it is properly secured to the skin. There should be a mild pressure on the acupressure point.

**Warnings:** Remove immediately if you feel discomfort or skin irritation occurs. Do not reuse patch once removed from the skin. For external use only. Do not ingest. Do not use on wounds or damaged skin. Ask a health professional before using if you have a health condition, any questions or concerns about your health. Do not use if pregnant or nursing. This booklet is no way intended to be medical advice. Individual results will vary.

Product to be stored at room temperature.

## The Clock / Cross Method

The Clock/Cross Method was developed to provide fast and effective results for non-specific, localized musculoskeletal pain.

Move through the locations shown in Fig. 1.0 to 1.6 until the pain has been substantially reduced and/or mobility has increased.

**Fig. 1.0**

**Fig. 1.1**

**Fig. 1.2**

**Fig. 1.3**

**Fig. 1.4**

**Fig. 1.5**

**Fig. 1.6**

This can be used on any area of the body. For specific patch protocols, please visit [www.lifewave.com](http://www.lifewave.com).

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LifeWave Europe Ltd., Raheen Ind. Est., Athenry, Co. Galway, H65 NW84, IRL.
LifeWave, Inc., 9775 Businesspark Ave, San Diego, CA 92131, USA.